


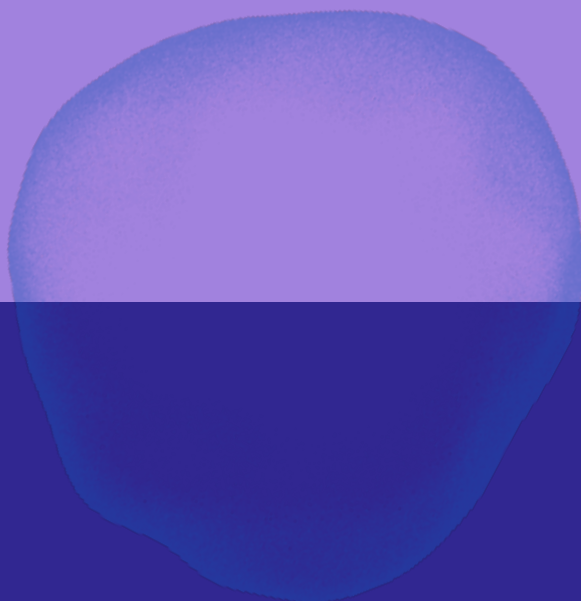

Quick Start Guide

VitalPatch® Adhesive Overlay



This guide provides step-by-step instructions
to use the adhesive overlay.

prolaio

Adhesive Overlay Indications for Use

The adhesive overlay is an optional accessory that was made specifically for the VitalPatch for use when the VitalPatch does not adhere for the entire monitoring period.


Adhesive Overlay Application

- For an instructional video, please see: <https://vitalconnect.com/docs/overlay-video.html>
- Note: It may be helpful to look in the mirror while applying the adhesive overlay.




Step 1: Prepare the Skin

If hair is present around the VitalPatch, trimming or shaving the area is advised. Clean the skin around the VitalPatch with an alcohol prep, avoiding the patch. Allow the skin to dry.

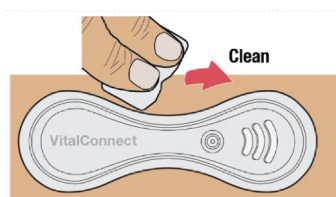


Step 2: Bend Tab

Find the notched corner on the adhesive and bend it up about a half inch to create a bent tab to help you remove it later.

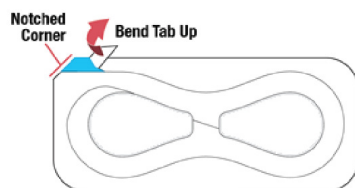


Step 3: Place Over VitalPatch

Place the adhesive overlay over the VitalPatch with the white side up, aligning the two overlay cutouts with the raised parts of the VitalPatch.

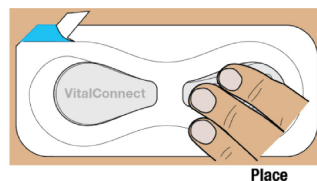


Step 4: Remove the First Blue Liner

Once the adhesive overlay is in place, hold one end with your hand and reach under the blue liner using your other hand. Slowly peel away and remove the blue liner. Press the adhesive overlay firmly onto the skin.

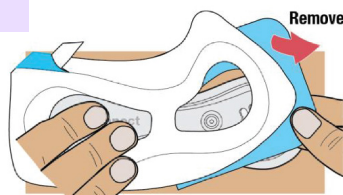


Step 5: Remove the Second Blue Liner

Repeat the previous step for the remaining blue liner.

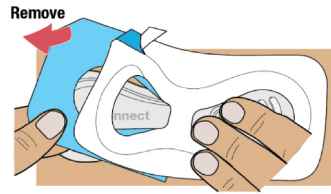


Step 6: Peel Off The White Liner

Press down on the adhesive overlay to adhere it to the skin, then peel off the top white liner starting at the bent tab while pressing the other hand on the VitalPatch.

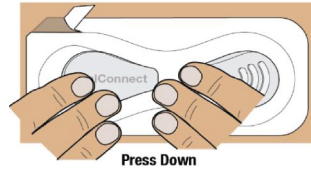



Step 7: Seal Edges

Press down on the adhesive overlay to seal the edges on your skin.

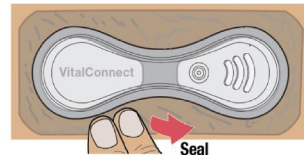

Removing the Adhesive Overlay

- Gently grab the corner of the overlay and pull it horizontally to help release it off the skin.
- Continue to peel off the adhesive overlay and the VitalPatch while gently sweeping the adhesive remover pad under the device. (See the VitalPatch remover instructions in the Prolaio Phone and VitalPatch® Quick Start Guide.)

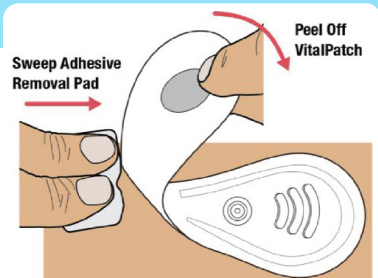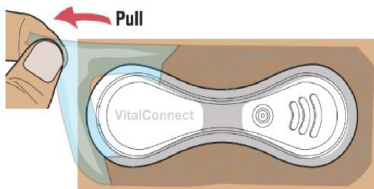

Adhesive Overlay Warnings and Precautions

The adhesive may cause adverse skin reactions, and you should seek medical attention if:

- You have a severe adverse event.
- You have an allergic reaction persisting beyond 2-3 days.

If discomfort or irritation happens, the overlay should be removed.

Icon and Description



Manufacturer



Re-use is not allowed



Medical Device



Caution, consult documents



Batch Code



Read instructions for use



Catalog Number



Not to be used in case
package is damaged

prolaio

Contact us

(855) 869-9054

Monday-Friday, 9am - 5pm CST
(excluding major holidays)

support@prolaio.com

230 W. Monroe, Suite 2560
Chicago, IL 60606 USA

